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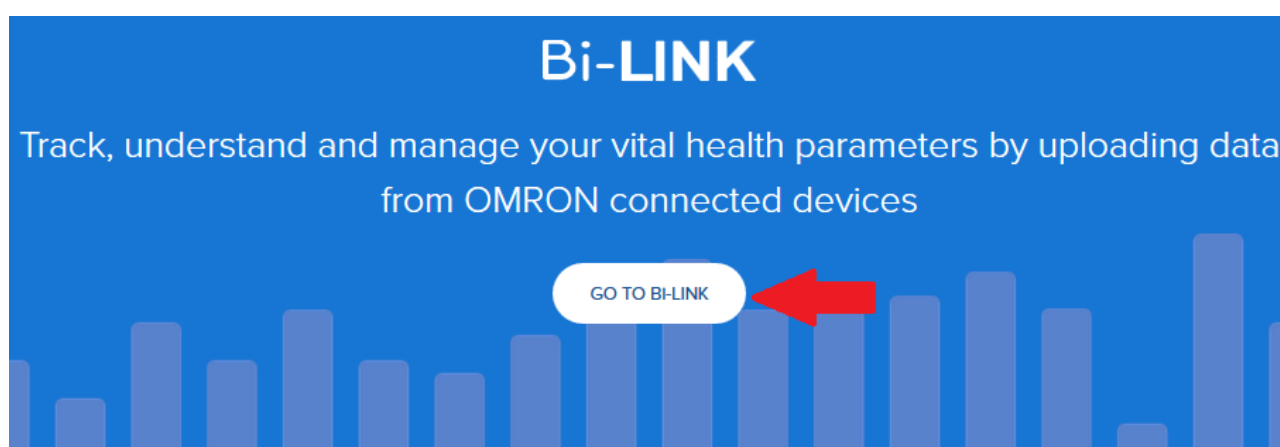
(tonometr OMRON M6 Comfort IT, OMRON M3 IT)

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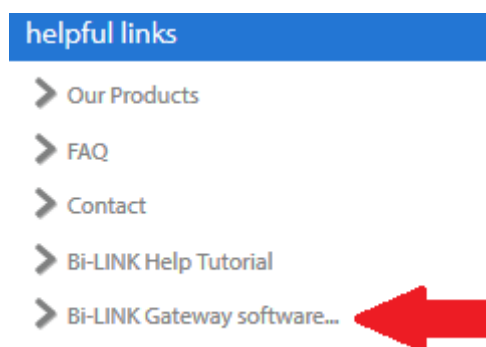
Krok 2

DOWNLOAD

How do I install the Bi-LINK Gateway desktop application to upload my data?

To upload your device data, please download and install the Bi-LINK Gateway utility. This utility is available for Windows PCs and it can be downloaded from: [Bi-LINK Gateway software download](#) 

I've Downloaded Bi-LINK Gateway, how do I upload my data?

Now that the Bi-LINK Gateway utility has been installed, login with your Bi-LINK account information. Once logged in, simply select your device.

Your device is now ready to upload. Simply insert your device data into your USB reader and select the matching device name from your personal Gateway library. Once you've chosen the device you wish to upload from, click "Upload Data" and "Start Upload". The Gateway utility will guide you through the device upload process.

I uploaded my data using the Bi-LINK Gateway Utility, how do I view my data?

Return to the Bi-LINK web application and click the Dashboard Tab. Note: there may be a brief delay after uploading before your data is available on the web.

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The screenshot shows the login interface of the Bi-LINK Gateway (version 2.2.0.0). The window has a blue header with the Omron logo on the left and a 'Přihlásit se' button on the right. Below the header, there is a left-pointing arrow button. The main area contains two input fields: 'Uživatelské jméno' (Username) and 'Heslo' (Password). Below these fields is a checkbox with a checkmark icon and the text 'Zapamatovat si mě' (Remember me). A large blue button labeled 'přihlásit se' (Log in) is positioned below the checkbox. At the bottom, there is a link that says 'Zapomněli jste heslo?' (Forgot your password?).

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The screenshot shows the device selection interface of the Bi-LINK Gateway (version 2.2.0.0). The window has a blue header with the Omron logo on the left and a 'Zvolit přístroj' (Select device) button with a dropdown arrow on the right. Below the header, there is a list of four blood pressure monitors, each with a circular icon showing '135' over '85' and a right-pointing arrow button. The devices listed are: 'M3 IT Blood Pressure Monitor' (HEM-7131U-E), 'M400 IT Blood Pressure Monitor' (HEM-7131U-D), 'M6 Comfort IT Blood Pressure Monitor' (HEM-7322U-E), and 'M500 IT Blood Pressure Monitor' (HEM-7322U-D). A vertical scrollbar is visible on the right side of the list.

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